

PATIENT CARE & HANDLING

Your custom contact lenses are coated with Tangible® Hydra-PEG® technology to enhance their wettability. Tangible Hydra-PEG has been shown to increase surface water retention and lubricity, and minimize deposits on various contact lens materials.

To care for your Tangible Hydra-PEG coated lenses, follow these instructions:

CAREFUL! LENSES ARE SLIPPERY

Plug your drain to avoid losing your lenses.

DO NOT USE TAP WATER

Only use buffered, preservative-free sterile saline solution to rinse your lenses.

DO NOT USE ENZYMATIC, ABRASIVE OR ALCOHOL-BASED CLEANERS

They may damage the Tangible Hydra-PEG coating.

Always follow the solution and lens care recommendations provided by your eye care practitioner.

For more information, visit tangiblescience.com/care-tips

